# PreCold® LEIÐBEININGAR UM NOTKUN

## LÝSING Á TÆKI

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# PreCold® INSTRUCTIONS FOR USE

## DEVICE DESCRIPTION

PreCold® is a medical device that protects against the common cold. PreCold® has fast onset of action and is easy to use. PreCold® works by coating the lining of the mucous membrane, creating a protective barrier against the common cold viruses. The barrier acts osmotically on the cold viruses, trapping the viruses and deactivates the ability of the viruses to bind into human cells. The PreCold® barrier protects the mouth and throat, helping the body to remove deactivated virus in a natural way.

## DESCRIPTION OF DISEASE: THE COMMON COLD

The common cold is one of the most common infectious diseases caused by viruses that enter the body via the nose, mouth and throat<sup>1</sup>. The common cold is caused by a diverse set of viruses where rhinoviruses are the major pathogens. The incubation time is approximately 48 hours, depending on the type of virus (range 0.5 to 4.5 days<sup>2</sup>). The immune system reacts to the common cold virus through an inflammatory reaction. The inflammatory reaction causes symptoms such as sore throat, cough, runny nose and sometimes fever and mild coughing.

- A common cold usually lasts 7-10 days<sup>1</sup>
- An adult usually has 2-5 common colds per year<sup>3</sup>
- A child usually has 7-10 common colds per year<sup>3</sup>

<sup>1</sup> Heikkinen, et al. "The common cold." The Lancet 361.9351 (2003): 51-59

<sup>2</sup> Lessler, Justin et al. Incubation periods of acute respiratory viral infections: a systematic review. The Lancet Infectious Diseases, Volume 9, Issue 5, 291-300.

<sup>3</sup> Hull D, et al. Effects of creating a non-specific, virus-hostile environment in the nasopharynx on symptoms and duration of common cold. Acta Otorhinolaryngol Ital. 2007 Apr; 27(2): 73-7.

## INDICATION

To be used when exposed to common cold viruses or early on when common cold symptoms occur. PreCold® is suitable for adults and children over 4 years.

## CONTRAINDICATIONS

- No known contraindications.

## POSSIBLE SIDE EFFECTS

- No known side effects.

Should you experience untoward effects while using PreCold®, please consult a physician for advice.

## INTENDED USE

- PreCold® reduces the probability of catching a common cold.
- PreCold® can help shorten the duration of a common cold if used at an early stage of the infection.
- PreCold® alleviates common cold symptoms and soothes sore throat.

## MODE OF ACTION

The PreCold® barrier consists of two major ingredients that together provide the main effective result.

## GLYCEROL + COD TRYPSIN

Traps: Glycerol creates an osmotically active barrier that traps viruses.

Deactivates: Trypsin inhibits the viruses' ability to bond, deactivating their ability to infect cells.

Protects: The PreCold® barrier protects the oral cavity and throat.

TRAPS

DEACTIVATES

PROTECTS

## PRECAUTIONS

- Do not use PreCold® if you are hypersensitive/allergic to any of the ingredients.
- Do not inhale when applying the spray since it may cause transient asthma-like symptoms, such as coughing and hoarseness.
- Consult a physician if you have a high fever or if your symptoms are more severe than those of a normal common cold; they could be due to something else.
- Keep out of reach of children, to prevent accidents with small parts.
- Do not use the product for more than 30 consecutive days, as there is no clinical information about long term use.

## PREGNANCY AND BREASTFEEDING

There is no clinical information about the use of PreCold® during pregnancy and breastfeeding; please consult a physician before use.

## THE PRECOLD® PACKAGE CONTAINS

- Mouth Spray consisting of bottle, pump, nozzle and protective cap.
- Solution containing glycerol, water, a natural enzyme – cod trypsin, ethanol (<1%), calcium chloride, trometamol and menthol. The solution is sugar-free and contains no preservatives, gluten or lactose.
- This instructions for use.

## DIRECTIONS FOR USE

**How to use PreCold® to help prevent a common cold:**

If you do not have any symptoms of a common cold but are likely to be exposed to the common cold virus, for example when someone near you has a common cold.

1. Open your mouth and aim the nozzle towards your throat (Figure 1).
2. Press down on the pump and spray 2 times (1 dose) every second hour up to 6 times daily during the time you are exposed to the common cold virus.

## FIGURE 1

## How to use PreCold® to help shorten the duration of a common cold:

Start using PreCold® as soon as possible after noticing symptoms of a common cold. Continue to use it until the symptoms are relieved.

1. Open your mouth and aim the nozzle towards your throat (Figure 1).
2. Press on the pump and spray 2 times (1 dose) every second hour up to 6 times daily.

## CARE AND STORAGE

PreCold® must be used before the expiration date indicated on the product label and package. Store PreCold® at room temperature, max 25°C.

## EXPLANATION OF SYMBOLS

	Consult instructions for use		Batch code
	Upper limit of temperature		Catalogue Number
	Use by date		Manufacturer
			Caution, consult instructions for use

**MANUFACTURED BY**  
Enzymatica AB  
IDEON Science Park  
SE-223 70 Lund, Sweden  
info@enzymatica.com  
enzymatica.com

**CZ-MS06002001S**  
PreCold® 20 ml

MEDICAL DEVICE